

Mepilex® Border

THE EFFECTIVE ABSORBENT ALL-IN-ONE FOAM DRESSING

- Minimises pain and trauma at dressing changes^{1,2}
- Excellent and effective exudate management
- Safe fixation but still atraumatic removal
- Shower proof

Polyurethane backing film

- Breathable
- Water proof
- Bacteria and viral barrier*

Safetac® layer

- Reduces pain and trauma before, at and after dressing change¹
- Does not adhere to the moist wound bed but to dry skin only
- Seals the wound margins and reduces risk of maceration³

Multi-layered absorbent foam pad

- Polyurethane foam to quick exudate absorption
- Fibre layer for maximum retention
- Non-woven spreading layer for optimising the absorption capacity and leakage prevention




* microbes larger than 25 nm

Safetac® technology

Safetac® technology. Less pain and less trauma.

Safetac is a patented adhesive technology that minimises pain to patients and trauma to wounds. Safetac technology is available exclusively on Mölnlycke Health Care dressings, including Mepilex® wound care dressings, Mepitel®, Mepiform® and Mepitac®.

For more information visit www.safetac.com

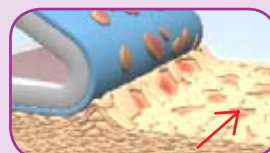


Skin stripping occurs with traditional adhesive




No skin stripping occurs with Safetac technology

How to use Mepilex® Border

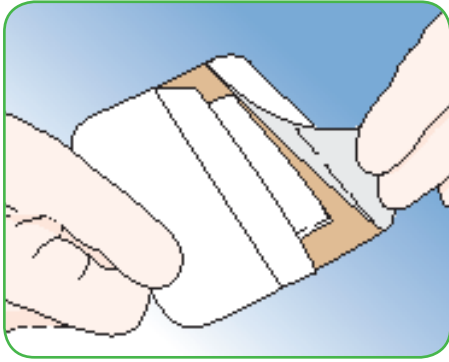

Clean the wound area.
Remove the release film.

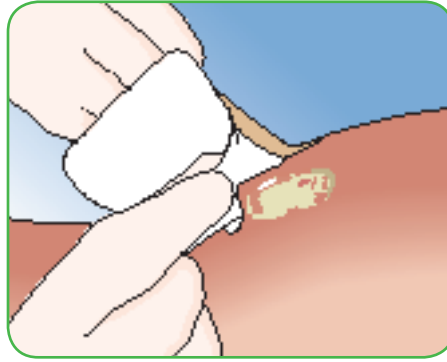

Apply the adherent side to the wound.
Do not stretch.

How Mepilex Border works

Mepilex Border is an all-in-one foam dressing that effectively absorbs and retains exudate and maintains a moist wound environment. The Safetac layer seals the wound edges, preventing the exudate to leak onto the surrounding skin, thus minimising the risk for maceration. The Safetac layer ensures that the dressing can be changed without damaging the wound or surrounding skin or exposing the patient to additional pain.

Benefits of Mepilex Border

- Minimises pain and trauma at dressing changes
- Self-adherent - no secondary fixation needed
- Shower proof
- Can be used under compression bandages
- Promotes patient comfort during wear
- Can remain in place for several days depending on the condition of the wound
- Can be lifted and adjusted without losing its adherent properties
- Low potential for skin irritation and allergy

Areas of use

Mepilex Border is designed for a wide range of exuding wounds such as leg and foot ulcers, pressure ulcers and traumatic wounds, e.g. skin tears and secondary healing wounds.

Note

In case of clinical signs of infection the use of Mepilex Border may be continued if proper infection treatment is initiated.



MEPILEX BORDER ASSORTMENT (Sterile packed)

Art. no	Size cm	Size inch cont.	Pcs/shelf cont.	Pcs/transp. cont.
295200	7,5 x 7,5	3 x 3	5	70
295300	10 x 10	4 x 4	5	50
295400	15 x 15	6 x 6	5	50
295600	15 x 20	6 x 8	5	45
Surgical sizes				
295800	10 x 20	4 x 8	5	35
295850	10 x 25	4 x 10	5	35
295900	10 x 30	4 x 12	5	25
Mepilex Border Sacrum				
282000	18 x 18	7 x 7	5	40
282400	23 x 23	9 x 9	5	25