



THE EFFECTIVE ABSORBENT FOAM DRESSING


- Minimises pain and trauma at dressing changes^{1,2}
- Gentle adherence with better stay-on-ability and comfort
- Does not slip under dressing retention and can easily be cut to size*

Safetac[®] layer

- Reduces pain and trauma before, at and after dressing change¹
- Does not adhere to the moist wound bed but to dry skin only
- Seals the wound margins and reduces risk of maceration³

Polyurethane foam pad

- Absorbs exudate
- Works well under compression
- Comfortable

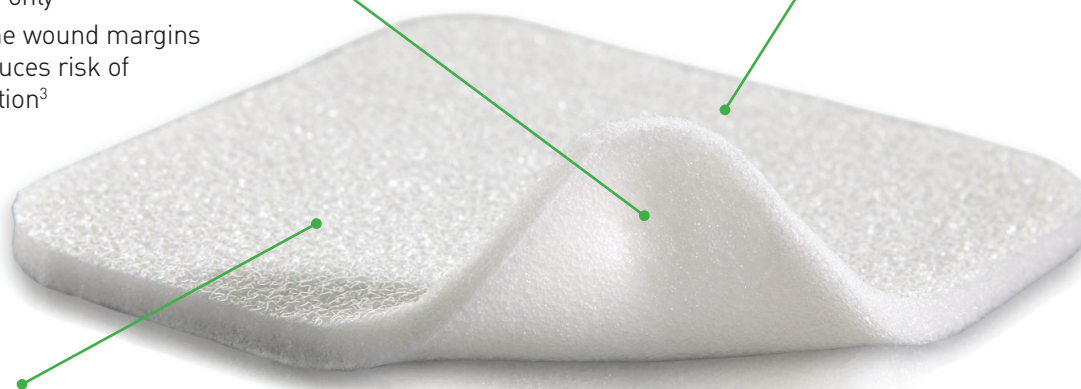

Polyurethane backing film

- Breathable
- Water proof
- Viral and bacteria proof*

Safetac[®]
TECHNOLOGY

* microbes larger than 25 nm


Safetac[®] technology

*Mepilex does not need to be cut to the size of the wound when compression is used


Safetac[®] technology. Less pain and less trauma.

Safetac is a patented adhesive technology that minimises pain to patients and trauma to wounds. Safetac technology is available exclusively on Mölnlycke Health Care dressings, including Mepilex[®] wound care dressings, Mepitel[®], Mepiform[®] and Mepitac[®].

For more information visit www.safetac.com

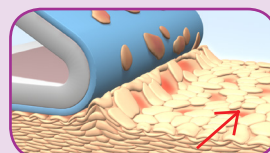


Skin stripping occurs with traditional adhesive

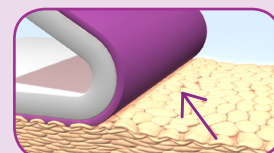

No skin stripping occurs with Safetac technology


How to use Mepilex®

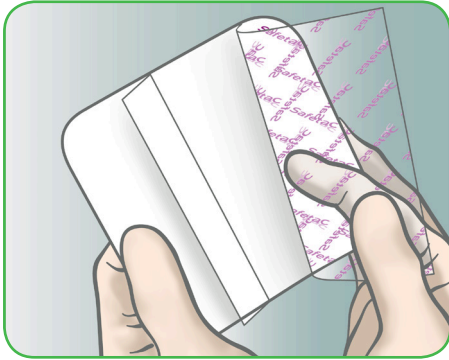

Clean the wound area.
Remove the release film.



Mepilex should overlap the wound bed by at least 2 cm onto the surrounding skin.
Apply the adherent side to the wound.
Do not stretch.

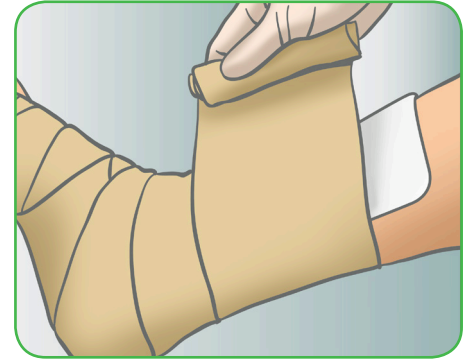

When necessary, secure Mepilex with a bandage or other fixation.

How Mepilex works

Mepilex is a soft and highly conformable foam dressing that absorbs exudate and maintains a moist wound environment. The Safetac layer seals the wound edges, preventing the exudate to leak onto the surrounding skin, thus minimising the risk for maceration. The Safetac layer ensures that the dressing can be changed without damaging the wound or surrounding skin or exposing the patient to additional pain.

Benefits of Mepilex

- Minimises pain and trauma at dressing changes
- Stays in place allowing for "hands-free" to facilitate application of compression or retention bandages
- Well suited to be used under compression bandages
- Can be cut to suit various wound shapes and difficult-to-dress locations
- Promotes patient comfort during wear
- Can remain in place for several days depending on the condition of the wound
- Can be lifted and adjusted without losing its adherent properties
- Low potential for skin irritation and allergy



Areas of use

Mepilex is designed for a wide range of exuding wounds such as leg and foot ulcers, pressure ulcers and traumatic wounds, e.g. skin tears and secondary healing wounds.

Note

In case of clinical signs of infection the use of Mepilex may be continued if proper infection treatment is initiated.



MEPILEX ASSORTMENT (sterile packed)

Art. no	Size cm	Size inch	Pcs/shelf cont.	Pcs/transp. cont.
294100	10 x 10	4 x 4	5	70
294000	12.5 x 12.5	5 x 5	5	50
294200	10 x 20	4 x 8	5	45
294300	15 x 15	6 x 6	5	25
294400	20 x 20	8 x 8	5	20
294500	20 x 50	8 x 20	2	12

MEPILEX HEEL ASSORTMENT

288100	13 x 20	5 x 8	5	25
288300	15 x 22	6 x 9	5	30